

Needle-Free KITS: NON-trochar



NON-trochar (2012-K; 12-16 Fr size)

Design

- DOUBLE pouched soft PVC drain
- Radiopaque line for confirmation of placement on X-ray
- Clear distance markings for accurate placement
- Multiple side drain holes for maximum efficiency
- End opening is same diameter as drain lumen for maximum drainage
- Drain material is the same as plain set (page 33)
- Available as a single item



NON-trochar 16 Fr; (2012-16K)

DETAILS: NON trochar set (2012-K)

- Sets (2012-K) are fully assembled and bonded - no assembly required
- C-clamp on drain for security and attachment point
- Generous drain tube length; drain length cannot be modified
- Place at routine thoracotomy closure, or use two-forcep technique for emergency placement
- Antegrade placement only
- Female luer connects to any luer-fitted device
- Connection is needle-free, making aspiration & kernal management much easier
 - ⇒ Valve closes automatically when detaching syringe or luer
 - ⇒ Self sealing means no need to cap the end to maintain sterility
 - ⇒ No risk of accidental opening



A swappable Needle-Free valve is used on all our chest drain kits.

Product Summary and Codes

Drain Type	Gauge Fr	Length (cm)		Pack Size	Product Code
		Total	Proximal ¹		
NON-trochar	12	30	20	1	DR2012-12K
NON-trochar	16	31	22	1	DR2012-16K
NON-trochar	20	45	35	1	DR2012-20K
NON-trochar	28	45	35	1	DR2012-28K

Note: 1. 'Proximal' refers to length from proximal most fenestration hole