

Disposable Silicone Foley Catheters

Instructions for Use

Specification

	6Fr,8Fr,10Fr,12Fr, 14Fr,16Fr,18Fr
	6Fr,8Fr,10Fr,12Fr, 14Fr,16Fr,18Fr, 20Fr,22Fr,24Fr,26Fr
	16Fr,18Fr,20Fr, 22Fr,24Fr,26Fr

Intended use

Disposable Silicone Foley Catheter is a thin, sterile tube inserted into the bladder to drain urine.

Method of use

1. Peel off the single package of catheter, confirm the catheter is applicable.
2. Generously lubricate tip and shaft of the catheter.
3. Carefully insert catheter tip into the bladder (normally indicated by urine draining flow), and then a further 3cm to ensure that balloon also insert urinary bladder..
4. Use a syringe without needle, inflate balloon with distilled water or saline.
5. For deflation, cut off inflation funnel, above the valve or using a syringe without needle push into valve to facilitate drainage.

Contraindications

No absolute contraindication.

Notice

1. Do not use if the package has been previously opened or damaged.
2. Can not be reused, or likely cause cross-infection.
3. Every access of the store room should keep clean and sterile.
4. The validity sterilization period is 36 months.